



DERMOCHOICE Application Manual

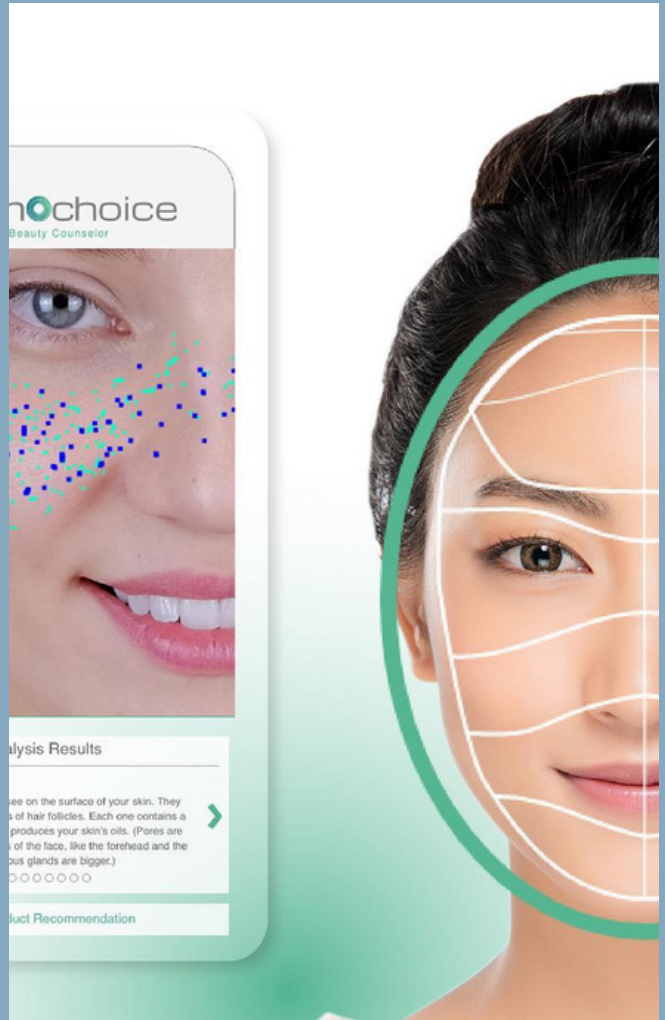

Analysing skin health using cameras from laptops, tablets, and smartphones, eliminating the need for a separate diagnostic device

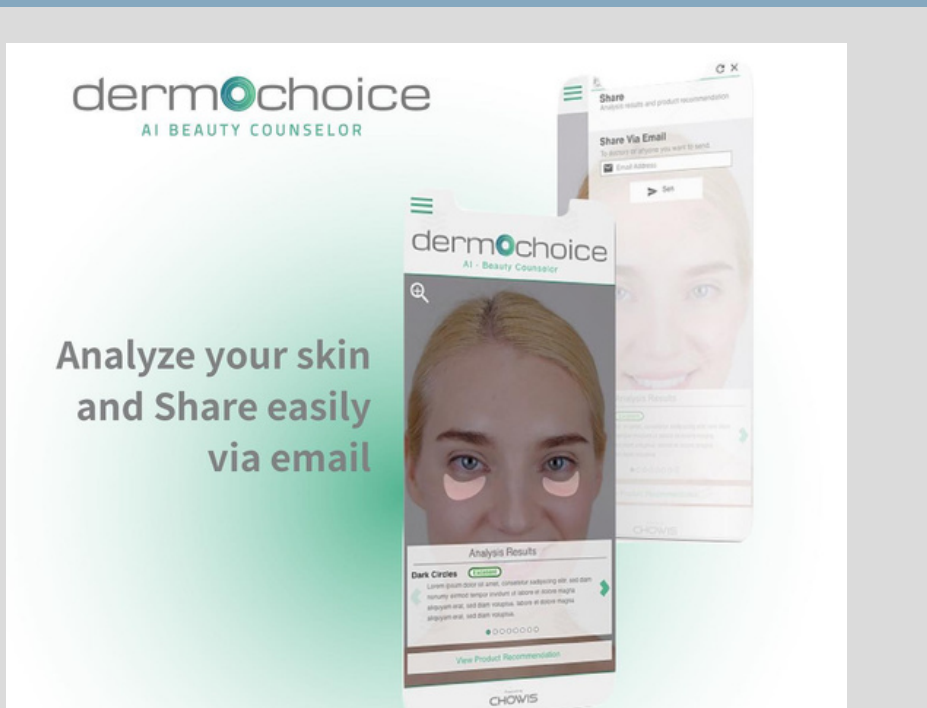

Analyze your skin
and Share easily
via email

Try it now!




CONTENTS

1. Open DermoChoice website	1
2. Input essential information	2
3. Guidelines walkthrough.....	3
4. Giving consent.....	4
5. Skin Analysis Steps.....	5-8
• Image Capture.....	5-6
• Questionnaire.....	7
• Analysis & Results.....	8
6. Contact us.....	9

OPEN DERMOCHOICE WEBSITE

Go to www.dermochoice.com or scan QR code to start Full Facial Analysis! Click "Let's begin"

Try it now!

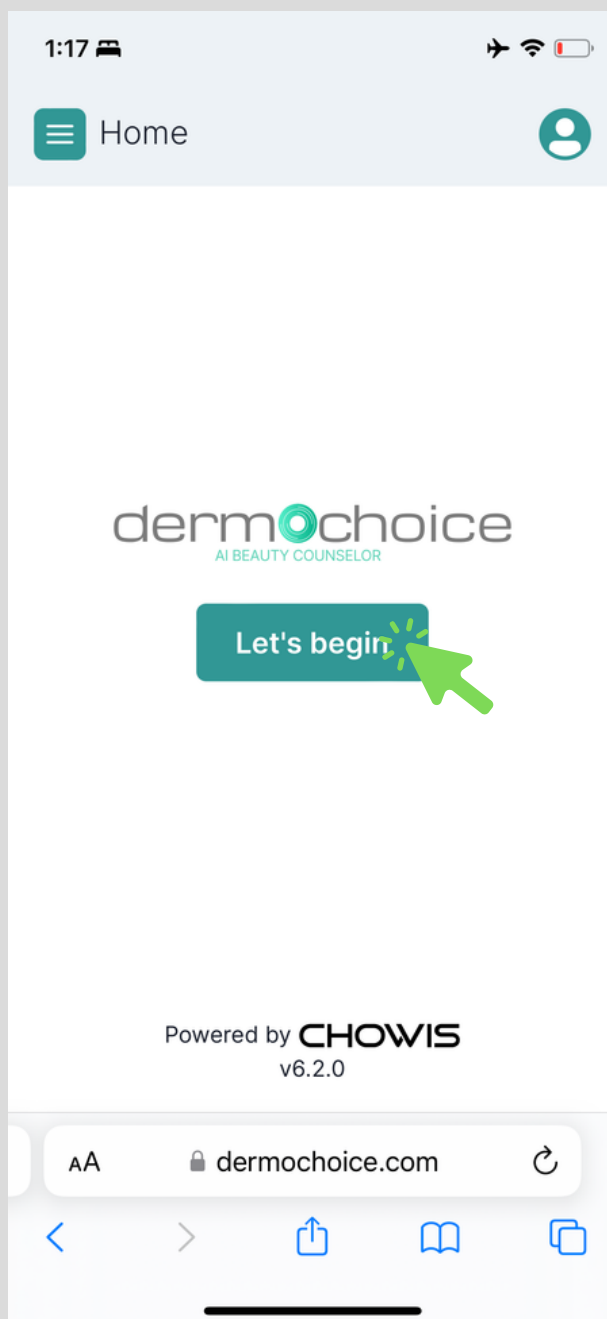

INPUT ESSENTIAL INFORMATION

- Fill in gender, birth, skin group, ethnicity information required for analysis, agree to privacy policy and click “Proceed”

1:17

Essential Information

Gender

Male Female

Birth Year *

Skin Group *

Ethnicity *

Proceed



1:17

Essential Information

Gender

Male Female

Birth Year *

1 9 9 4

Skin Group *

Ethnicity *

Asian East (Korea, Japan, Chi...

Proceed



GUIDELINES

- Before to move to the next step, please, view the Guideline how to take images correctly

The slider transitions every 5 seconds, or you also have the option to swipe through all the content to proceed to the next page.

Hello, Shivani!

By using advanced algorithms and big data, Dermochoice online skin analysis system can analyze multiple parameters in multiple platforms.

Accurate analysis results that will recommend specified products or treatments for your skin.

Full face analysis that shows previous results, helps you track your skin progress.



Swipe to see next slide

Guidelines



Fit your face directly to the camera



Please keep natural face expression while analyzing



Don't cover your face with your hand



Please remove your makeup

Guidelines



Take off the glasses and close your eyes while analyzing



Find a room with a good lighting



Please turn your silence mode off and turn up your volume

GIVING CONSENT

- Once these preparations are complete, the consent prompt will appear. Please, read the consent and if you agree, press “Proceed” to continue your skin analysis!

Consent

This app collects information and photos of your face and skin, along with description of skin health condition, concerns, and product recommendations.

This information is collected by Chowis Co., Ltd. and is required to the app to work. Your data will not be seen by other users nor provided to any third parties without your explicit consent.

I consent to Chowis Co., Ltd. processing my information to provide the app service to me as explained above.


Proceed



IMAGE CAPTURE

- Permit the application to access your device's camera & image capture
- The advanced AI within the application will instantly detect your face. A voice command will guide you through aligning your face within the circle, once it is aligned correctly, and background will turn to green, please, press “Start” button to capture images.

**Note: It will also give you an option, to manually capture the image.*

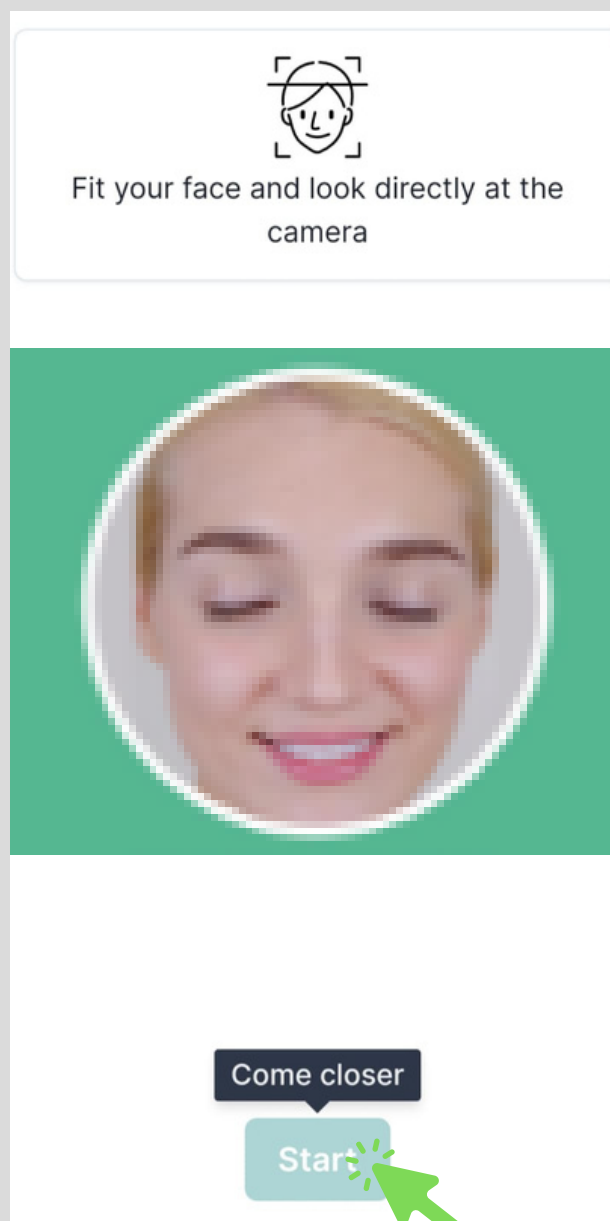


FACE DETECT

- The application will capture images of your face's front, left side, and right side for comprehensive analysis



Fit your face and look directly at the camera

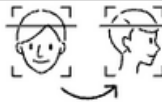

Slowly turn your face to the right



Slowly turn your face to the left




Fit your face and look directly at the camera



Slow turn your face to the right



Slow turn your face to the left

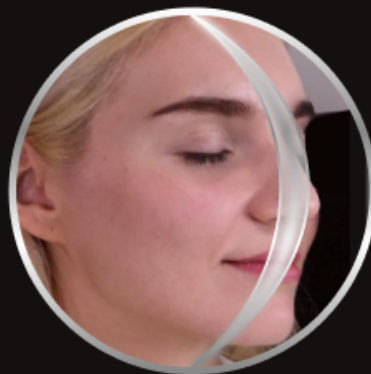

QUESTIONNAIRE

- User have the choice to answer these questions or skip them.
- By answering all the questions, DermoChoice will provide more accurate result!

Questionnaire

This questionnaire contains items that may take a few minutes to finish. Please select how you want to proceed


I'd like to complete this questionnaire and take advantage of full analysis

I want to answer all required questions only

I want to skip all questions for now

ANALYSIS & RESULTS

- DermoChoice will give the result of 8 different measurements: Spots, Wrinkles, Pores, Radiance, Elasticity, Dark Circles, Dullness, Oiliness.
- Skin Health result, with scale from 0 to 100, where higher index represents better skin health.
- On the bottom of the result, user can review main analyzed images.

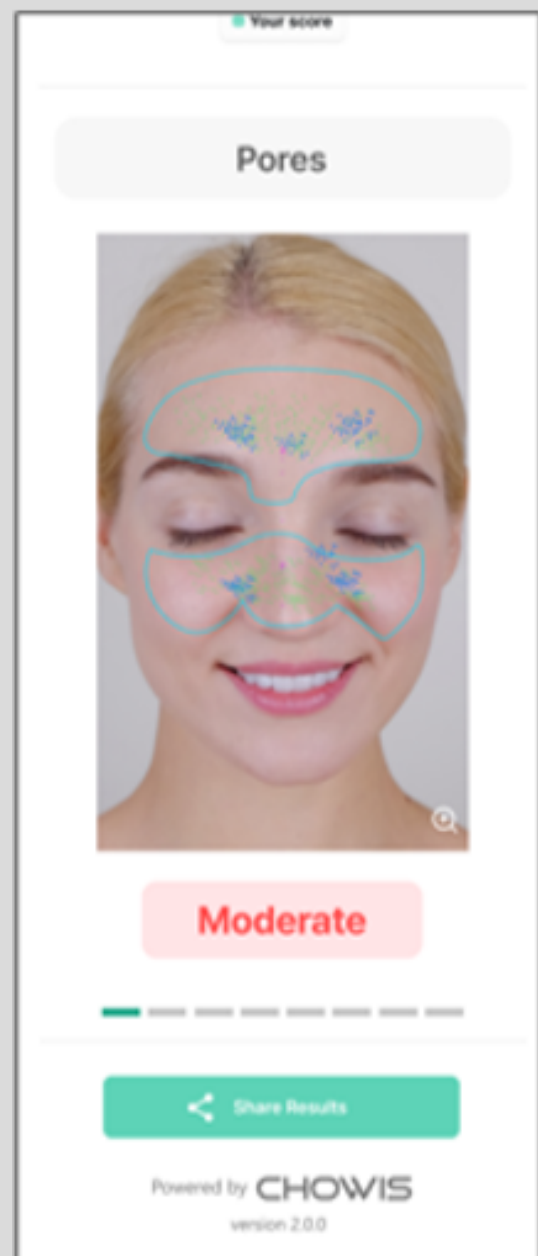

CONTACT US



CHOWIS
YOUR CHOICE OF DIAGNOSIS

CONTACT US



Facebook · Instagram @chowisofficial

Homepage www.chowis.com

Skype ID Chowisofficial

CONTACT US:

Email: chowis@chowis.com

Skype ID: chowisofficial

Chowis Co., Ltd.

Homepage: www.chowis.com

Tel: +82.31.706.7950 / Fax: +82.31.272.7950

Address: A-1301-2, 184,
Jungbu-daero Giheung-gu,
Yongin-si Gyeonggi-do,
Republic of Korea, [17095]