

# CHOWIS

YOUR CHOICE OF DIAGNOSIS



## myBeauty Kiosk

## Full Manual



Let's Begin



Ver. 1.0.0

# CONTENTS

- Device Overview. .... 1
- Package Inclusion..... 1
- How to Use the Device..... 2
- How to Login.....3
- Account Set Up. .... 4
- How to Register for Skin ..... 5
- Skin Analysis..... 6-7
- Skin Questionnaires. .... 8
- Results..... 9
- Share Results..... 10
- Hair Analysis Account Set up ..... 11
- How to Register for Hair..... 12
- Hair Analysis .....13-14
- Hair Questionnaires ..... 15
- Results..... 16
- Settings.....17
- FAQ..... 18
- Appendix..... 19
- Specifications..... 20
- Contact Us..... 20

# DEVICE OVERVIEW

## DEVICE OVERVIEW




1. 1.21 in Tablet  
2. Diagnostic Device

3. UV Stand  
4. PCB Dock

## PACKAGING INCLUSIONS



myBeauty Kiosk

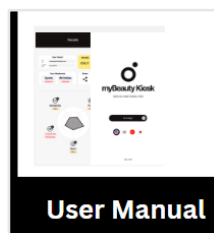

Quick Manual



USB Cable

# HOW TO USE DEVICE

**Please make sure to read these instructions in the manual carefully before using the device for safety and proper use.**

**Do NOT touch or insert any items into the optical unit/lens.**

Putting foreign substances inside the optical unit or touching the inner part of the lens can affect image quality causing the images to become blurred or unclear. In case there is dust on the lens, use the provided air blower to remove it. Also, do NOT put fingers or hands or other substances into the optical unit/lens. Potential damage can occur to the handset if attempted.

**Do NOT look directly into the light of the optical unit.**

The optical unit has multiple LEDs which can emit high brightness. Do not look into the light directly to prevent damaging the eyesight.

**Do NOT let conductive materials/substances come into contact with the chargersocket or the sensor of the handset.**

Do NOT let conductive material (scrap metal, lead etc.) come into contact or get inside of the sensor or the charger socket of the handset. If something seems wrong, stop using the handset and inquire at the retailer or customer support center.

**Use the device at temperatures between 5c° ~ 40c°.**

Proper storage is required for the device. Potential damage can occur on the handset such as errors or abnormal battery discharge in case the handset is used or stored in too hot or too cold places.

# HOW TO LOGIN

## LOGIN STEPS

1. As an administrator, click on Sign Up to create an account and log in to the app.
2. Register the device after logging in.
3. Enter the Device Optic Number and SSID Number once registered.
4. After completing these steps, proceed with the analysis.

The screenshot shows the 'Login' screen of the CHOWIS app. At the top, the CHOWIS logo is displayed with the tagline 'YOUR CHOICE OF ANALYSIS'. Below the logo, the text 'Administrator' is centered. There are two input fields: 'Email address' and 'Password'. Below the password field, there are links for 'Forgot Password?' and 'Not a member? SIGN UP'. A 'Register' button is located at the bottom of the form.

The screenshot shows the 'Administrator Account Registration' screen. It features three input fields for registration details. Below the fields, there are three checkboxes with associated text: 'I agree that my IP (real address) will be used to monitor device status and maintain my access to services from CHOWIS Co., Ltd.', 'I agree that my E-mail address will be used to receive the license confirmation message from CHOWIS Co., Ltd.', and 'I agree to the PRIVACY POLICY'. A 'Register' button is at the bottom.




The screenshot shows the 'Login' screen with the email 'sampleemail@gmail.com' entered. Under the heading 'Device Details', there are two input fields: 'Optic Number' and 'SSID Number'. Below these fields is a checkbox labeled 'I agree to the PRIVACY POLICY'. A 'Done' button is at the bottom.

The screenshot shows a success message on the CHOWIS app. A green checkmark is displayed above the text 'Perfect! How you can go further.' Below this text is a 'Done!' button.

# ACCOUNT SET-UP


## - START SCREEN

1. Tap the screen to begin.
2. Choose your language from the options provided.
3. Press '**Start**' to move on.

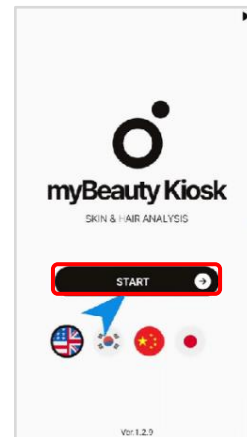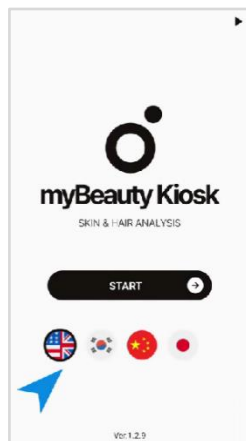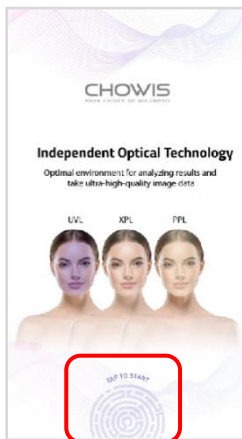

## ANALYSIS SELECTION







1. Select '**Skin**' or '**Hair**' analysis.
2. For this guide, pick '**Skin**.'

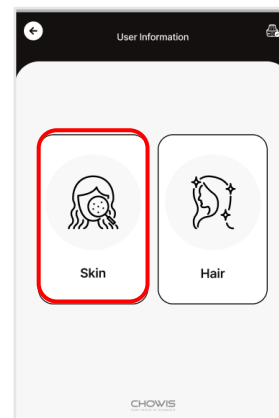

# USER REGISTRATION

## HOW TO REGISTER

1. If you are new, select 'No' when asked if you have an account.
2. Enter your birth year and gender, or skip this step.

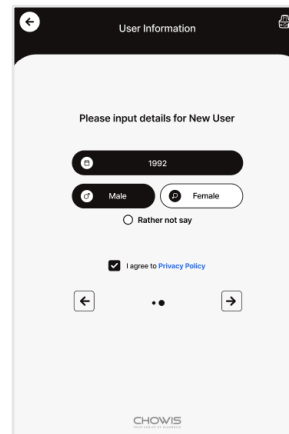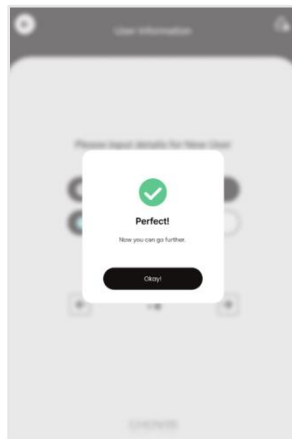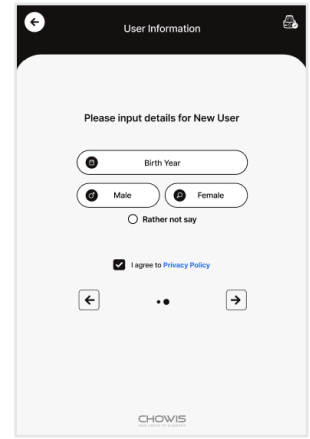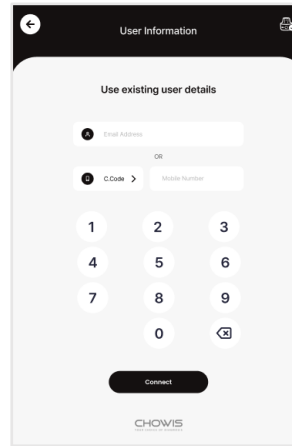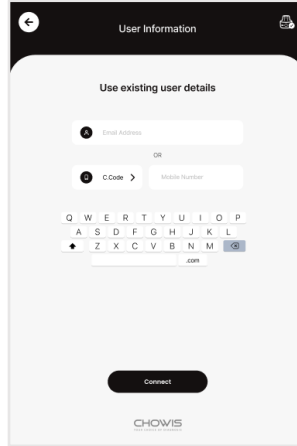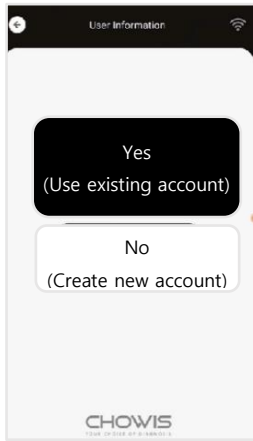


# SKIN ANALYSIS

## SKIN ANALYSIS - STEPS


### Skin Analysis:

1. A pop-up will appear with instructions on how to perform the analysis.
2. Place the camera gently on your skin.
3. Ensure the camera is covered by your skin.
4. Press the capture button and hold for **7 seconds**.
5. The camera will take three images.
6. After capturing, press **'Next'** to proceed or **'Retake'** if needed.

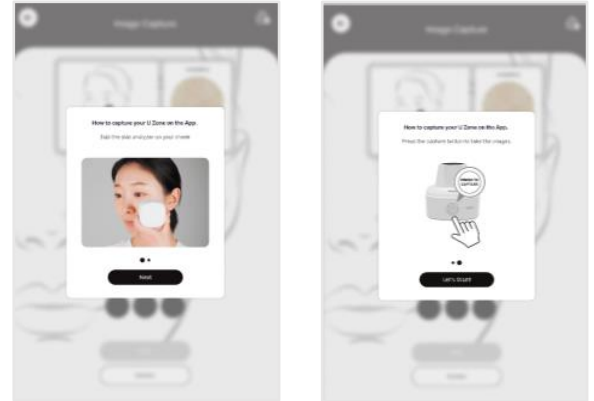

### Skin Analysis :

1. Capture U zone(cheek area).Place the analyzer on your cheek.
2. Press capture button and click on Next.
3. Press hold the device while taking 3 images.
4. When capture is complete, press **Next** to Proceed.
5. Click on **Retake**, if you want to redo the image capturing.


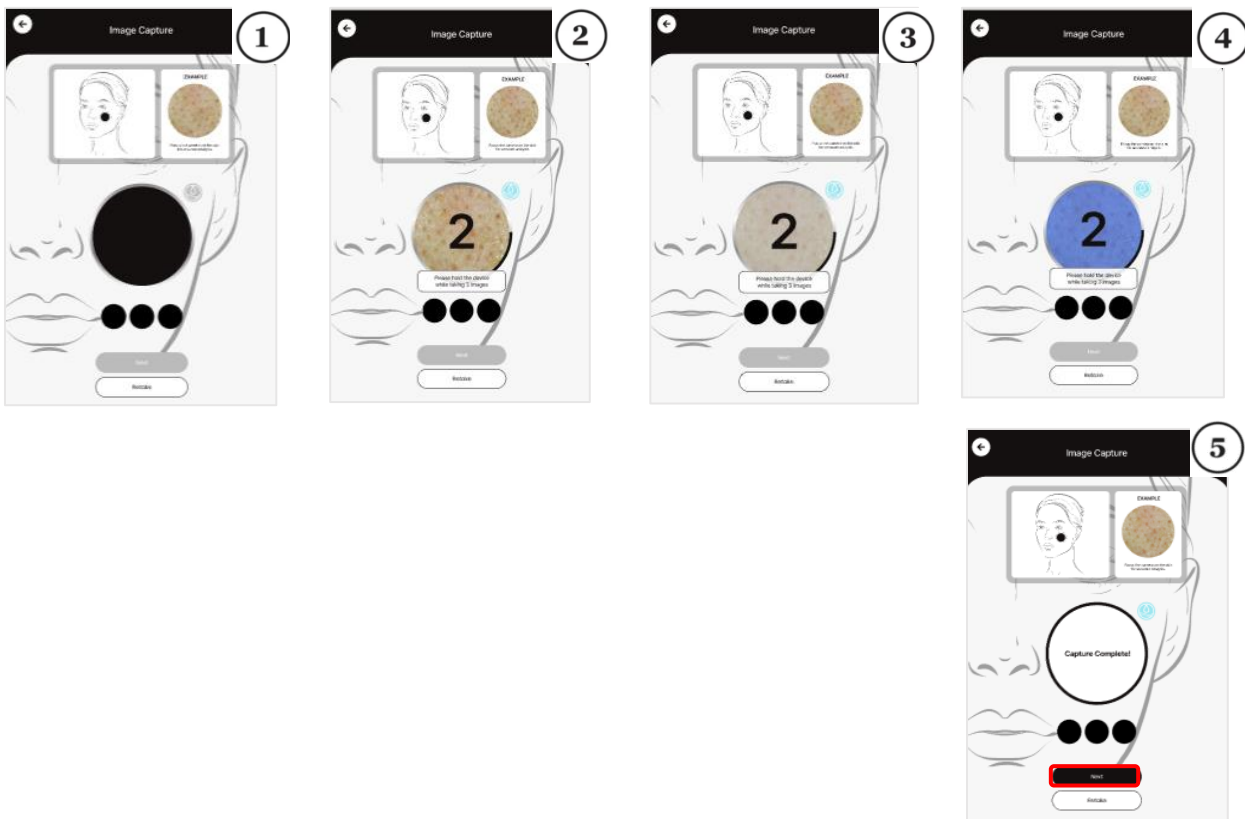




# SKIN ANALYSIS

## SKIN ANALYSIS - STEPS

1. You will receive another pop-up to do the next analysis.
2. Place the camera near the corner of the eye. You can take either left or right eye wrinkle image.
3. Ensure to close the eye.
4. Press the capture button and hold until it says '**Capture Complete**'.

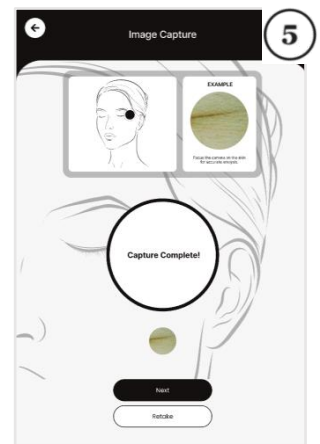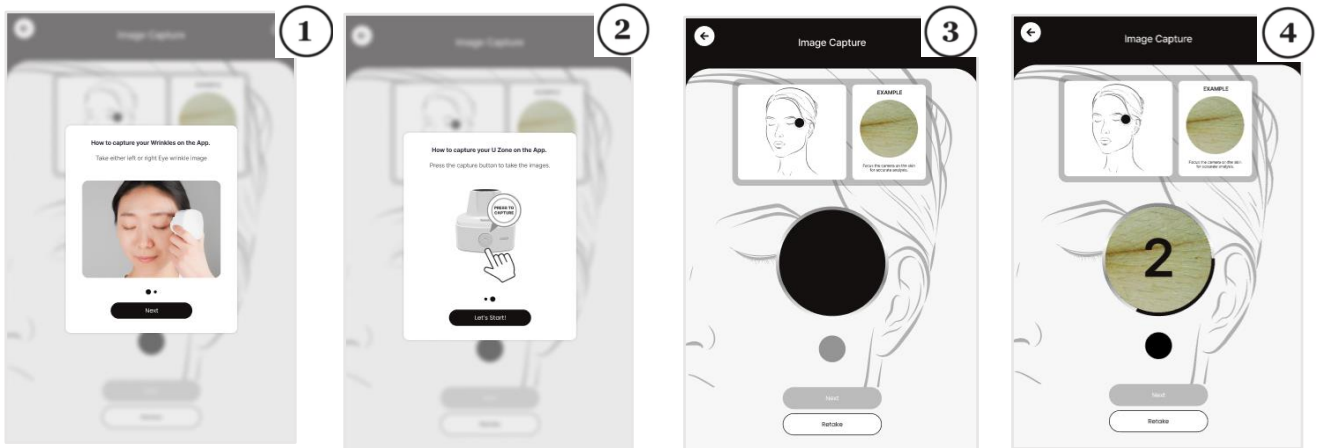

5. Put the camera back in the cradle.



# QUESTIONNAIRES - SKIN ANALYSIS

## QUESTIONNAIRES

1. The Questionnaires will pop up. You may answer them or skip the questionnaires.
2. After answering or skipping the questionnaire, you will be on "Results" page.

The screenshot shows the 'Skin Oiliness' questionnaire. At the top, there is a back arrow on the left and a camera icon on the right. The title 'Skin Oiliness' is centered. Below the title, the progress indicator '01 / 04' is shown. The question reads: 'With only makeup base and foundation applied on the face, how does do you look and feel after 2-3 hours without applying powder?'. There are five radio button options: 1. Flaky, 2. Smooth, 3. Oily (selected), 4. Greasy and slimy, and 5. I do not use makeup base or foundation. Below this, the progress indicator '02 / 04' is shown. The question reads: 'If you look closely at your face, how dilated are your pores? (compare to a size of a pin tip or bigger)'. There are three radio button options: 1. I can a few dilated pores, 2. Slightly more on the T-zone (forehead & nose) (selected), and 3. Many in different areas. At the bottom, there is a 'Skip Questionnaire' button highlighted with a red border.


The screenshot shows the 'Skin Pigmentation' questionnaire. At the top, there is a back arrow on the left and a camera icon on the right. The title 'Skin Pigmentation' is centered. Below the title, the progress indicator '05 / 05' is shown. The question reads: 'What is your natural skin complexion (without getting treatment or tanned skin)?'. There are four radio button options: 1. dark complexion, 2. medium toned complexion, 3. pale complexion (selected), and 4. very light complexion. In the background, there is a faint illustration of a face with a white dot on the forehead. At the bottom, there is a 'Next' button highlighted with a red border.

# RESULTS - SKIN ANALYSIS


## RESULTS

### Results Include:

- Skin Condition: Your current skin health.
- Skin Age: How your skin's age compares to your actual age.
- Weakness Box: Shows your top two skin issues.
- Spider Graph: A visual guide to your results (closer to the center is better).

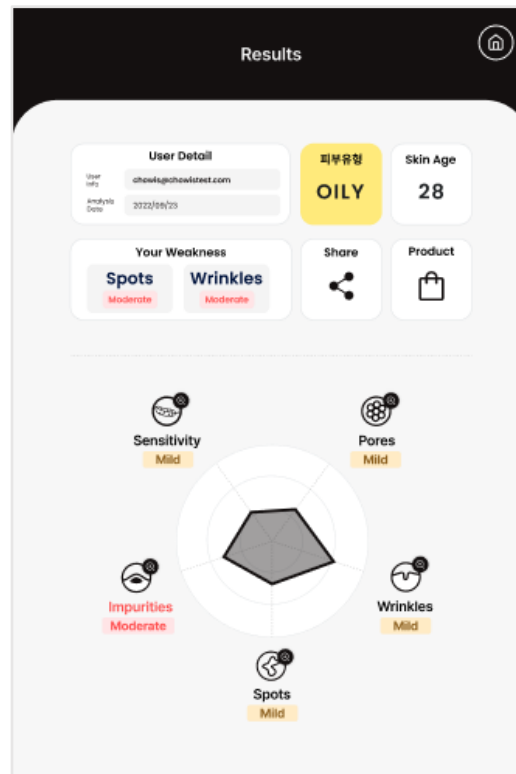


- View Images: Click on the parameter to see your captured images.

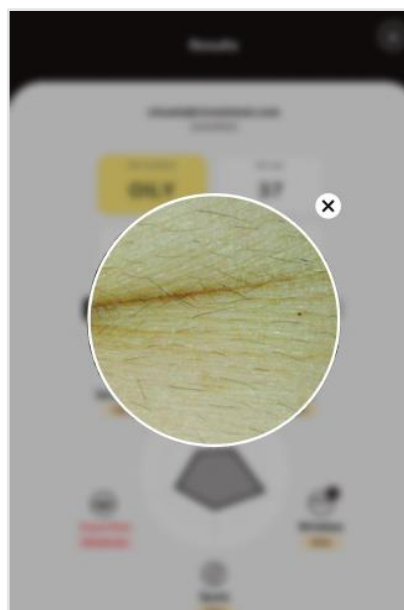

# SHARE RESULTS

## SHARE RESULTS

1. Click the share icon on the results page.
2. View the results once the survey is completed.

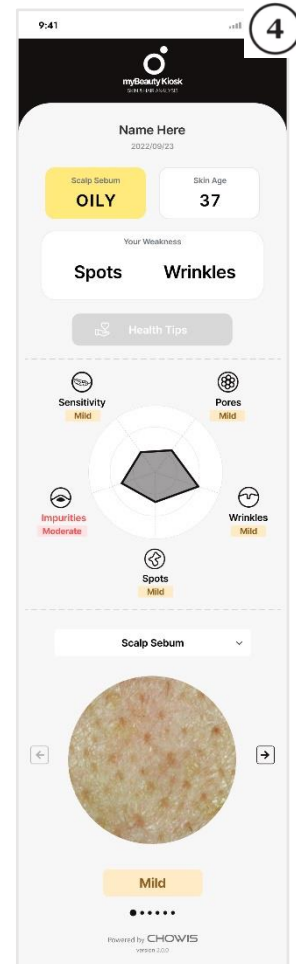


# ACCOUNT SET-UP

## HOW TO START


### - START SCREEN

1. Tap the screen to begin.
2. Choose your language from the options provided.
3. Press '**Start**' to move on.

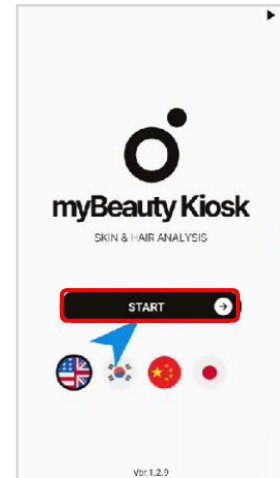

### Analysis Selection







1. Select '**Skin**' or '**Hair**' analysis.
2. For this guide, pick '**Hair**.'

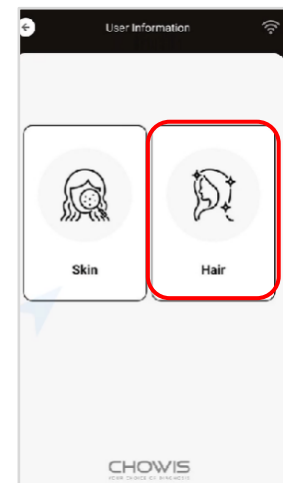

# USER REGISTRATION

## HOW TO REGISTER

1. If you're new, select 'No' when asked if you have an account.
2. Enter your birth year and gender, or skip this step.

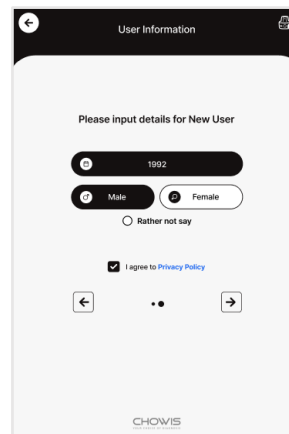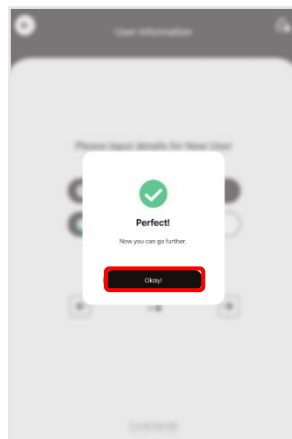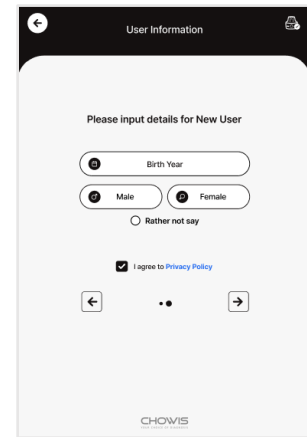
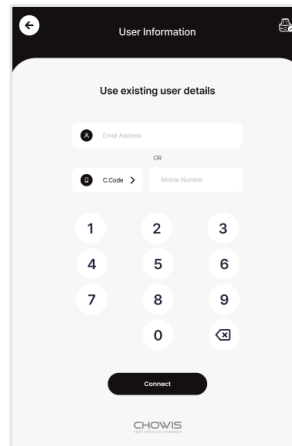
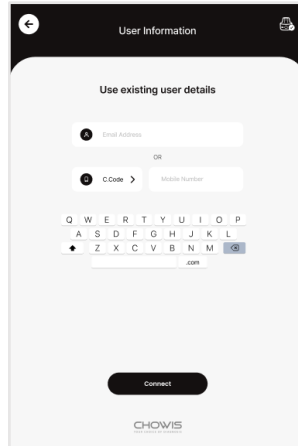
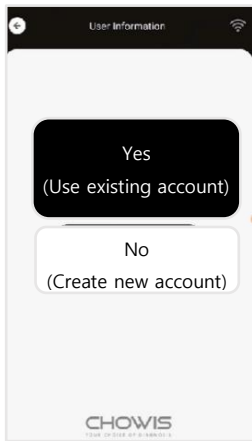


# HAIR ANALYSIS

## HAIR ANALYSIS - STEPS

### Steps

1. Focus the camera on your scalp for accurate analysis.
2. A pop-up window will guide you on how to capture the image.
3. Part your hair in the middle or on the side.
4. Pick up the Chowis Hair Analyzer and place it on your scalp.
5. Click "**Let's Start**".
6. Place the Chowis Hair Analyzer on your scalp and press the capture button.
7. Follow the guideline image at the top to place the analyzer on the frontal scalp area.
8. The image capture will start, and you'll see a timer.
9. Keep holding the device on your scalp while it takes two images.
10. Once done, a message saying "**Capture Complete**" will appear.
11. Click "**Next**" to proceed.

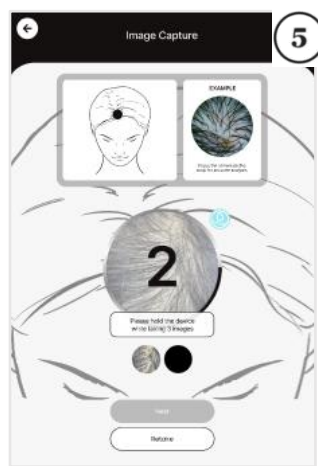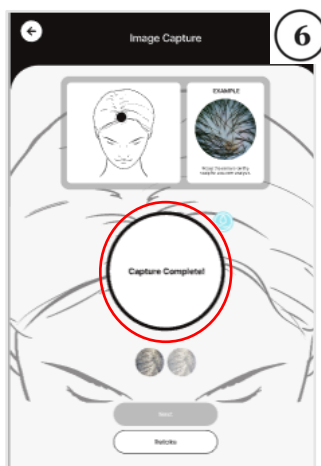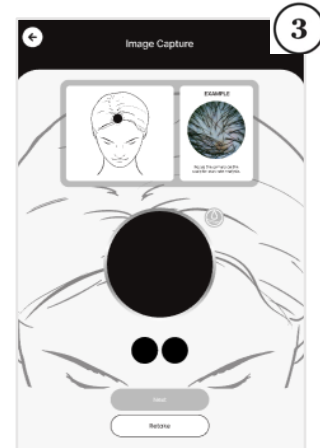
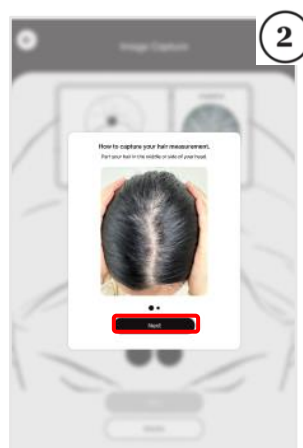


# HAIR ANALYSIS

## HAIR ANALYSIS - STEPS

### Steps

1. Focus the camera on your scalp for accurate analysis.
2. Follow the guideline image at the top to place the analyzer on the scalp area.
3. Pick up the Chowis Hair Analyzer and place it on your scalp.
4. Click "**Let's Start**".
5. Place the Chowis Hair Analyzer on your scalp and press the capture button.
6. The image capture will start, and you'll see a timer.
7. Keep holding the device on your scalp while it takes two images.
8. Once done, a message saying "**Capture Complete!**" will appear.
9. Click "**Next**" to proceed.

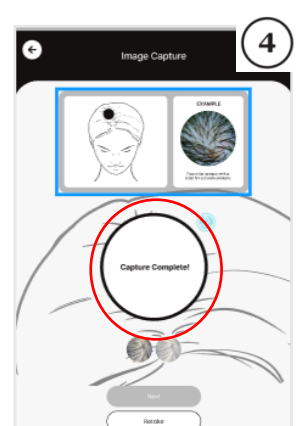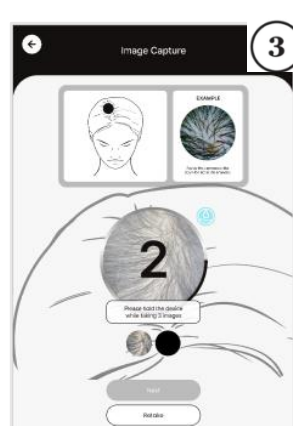



# HAIR ANALYSIS - QUESTIONNAIRES

## QUESTIONNAIRES

1. The Questionnaires will pop up. You may answer them or skip the questionnaires.
2. After answering or skipping the questionnaire, you will be on "**Results**" page.

The screenshot shows a mobile application interface for a questionnaire. At the top, there is a back arrow and the title "Questionnaire". The first question, labeled "01 / 10", asks "Which most closely describes the look of the skin on your face?". It has four radio button options: "My skin is oily and shiny all over", "My skin is cracked, peeling, or becomes itchy, irritated, or inflamed", "My skin is dry or normal in some areas and oily in others, such as the T-zone (nose, forehead, and chin)", and "Not too dry and not too oily skin". The second question, labeled "02 / 10", asks "Do you have any scalp problems?". It has four radio button options: "Oily and greasy scalp with yellow dandruff", "Dry and itchy scalp", "Combination", and "Normal". A "Skip Questionnaire" button is highlighted with a red box at the bottom.


The screenshot shows a mobile application interface for a questionnaire. At the top, there is a back arrow and the title "Questionnaire". The ninth question, labeled "09 / 10", asks "Compared with other scalp regions, are your frontal scalp and crown scalp especially itchy?". It has two radio button options: "Yes" and "No". The tenth question, labeled "10 / 10", asks "Did you notice a buildup, because of oiliness on your scalp?". It has two radio button options: "Yes" and "No". A "Next" button is highlighted with a red box at the bottom.

# RESULTS - HAIR ANALYSIS


## RESULTS

### Results Include:

- **Scalp Condition:** This section provides an overview of your scalp's current health and any hair damage. It helps you identify areas that may require attention.
- **Weakness Box:** Here, you'll find the top two scalp issues identified during the analysis. These are key areas to focus on for maintaining a healthy scalp.
- **Spider Graph:** The Spider Graph visually represents your scalp's condition across different concerns. The closer your results are to the center, the healthier your scalp is in those areas.

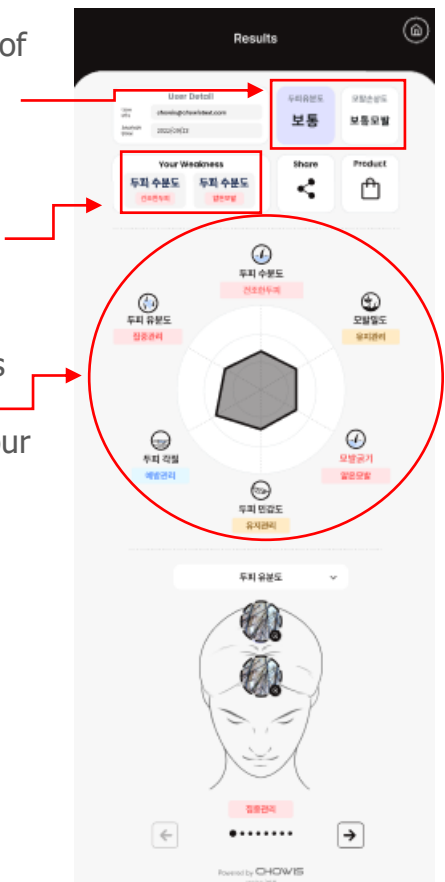


- **View Images:** Click on the parameter to see your captured images.

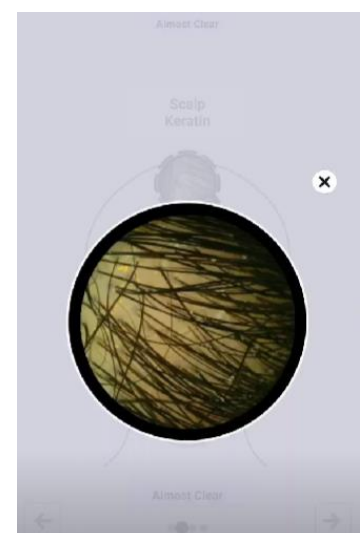


# SETTINGS


1. Under the Settings page, click on Device Setup to check if the device is registered.
2. Click on User Profile to review and edit your details.
3. Log out by clicking on Log Out from the app.

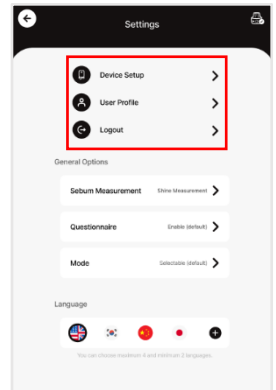


4. The questionnaires are enabled by default. If you want to disable them, click on the dropdown and select Disable.

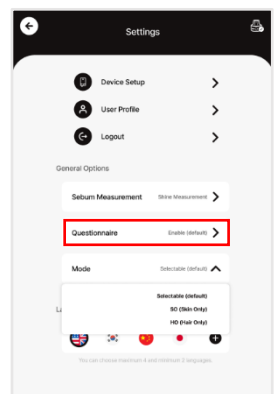

5. You can select the analysis mode. By default, both skin and hair analysis are enabled, but you can choose to perform only one (skin or hair) from the dropdown menu.

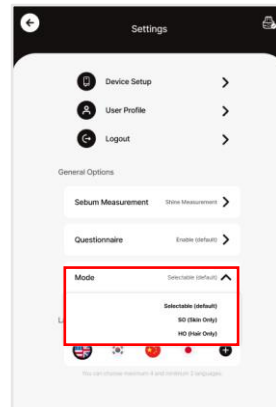

6. Click on the Language icon to select your preferred language from the list.

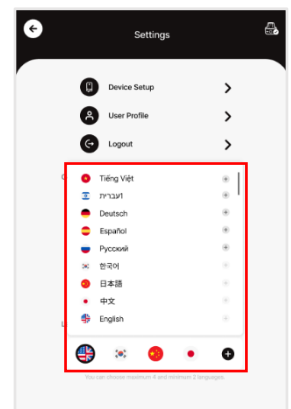

## DIAGNOSIS SYSTEM IN USE

### **USER AND CUSTOMER REGISTRATION**

Chowis' diagnostic system will first ask if you are a new user or have an existing account. If you are new, select 'No.' Fill in your email or phone number, or you can skip this part. Next, select your birth year and gender to complete the registration.

### **THE DEVICE FEELS HOT.**

As the optical unit/lens part emits a lot of LED light and long periods of continuous use can cause the device to heat up. If the heat is severe, immediately turn the device off and stop using it until it has cooled down. If the device is still hot even after a certain period of time has passed, please inquire at a customer support center. Also, a reason for over-heating could be when the device is left with power switched on during charging. Always remember to turn the handset power off before charging.

# APPENDIX

## APPENDIX: PROPER IMAGE CAPTURING

1. For Skin Analysis, focus the camera on the skin for accurate analysis.
2. For Skin Analysis, focus the camera on the skin for accurate analysis. Ensure no hair strands covering up the scalp area.
3. Keep holding the device on your scalp until both images are taken, then you'll see a "Capture Complete" message.

EXAMPLE



Focus the camera on the skin for accurate analysis.








EXAMPLE



Focus the camera on the scalp for accurate analysis.

# CONTACT US

## SPECIFICATION

 Power supply	Input: AC 100-240V 0.5A, 50/60Hz Output: DC 5V 2A
 Battery	Lithium Polymer Battery 3.7V 500mAh
 Product size	69.00 x 104.00 x 13.00 (mm)
 Product weight	52.5g - Device Only
 Analysis result	Clear, Almost Clear, Mild, Moderate, Severe
 Measurement Skin	Pores, Impurities, Wrinkles, Spots, Sensitivity, Moisture
 Measurement Hair	Hair Density, Keratin, Hair Thickness, Sensitivity, Scalp Moisture

## CONTACT

Email: [chowis@chowis.com](mailto:chowis@chowis.com)

Skype ID: **chowisofficial**

### **Chowis Co., Ltd.**

Homepage: [www.chowis.com](http://www.chowis.com)

Tel: +82.31.706.7950

Fax: +82.31.272.7950

Address: A-1301-2, 184, Jungbu-daero  
Giheung-gu, Yongin-si Gyeonggi-do,  
Republic of Korea, [17095]

